

# Staining Slides for IDEXX Digital Cytology\*



1. **Dip** 3 times
2. **Immerse** 30 seconds
3. **Tap** slide edge against absorbent material



1. **Dip** 3 times
2. **Immerse** 5 seconds
3. **Tap** slide edge against absorbent material



1. **Dip** 3 times
2. **Immerse** 5 seconds
3. **Tap** slide edge against absorbent material



- Dip** 5–10 times to remove excess stain



1. **Dip** 5–10 times to remove excess stain
2. **Air dry** using the “cool” fan setting