**SNAP Feline proBNP Test Kit**

For veterinary use only.

**SNAP\* Feline proBNP**

In vitro test for the measurement of circulating NTproBNP in feline serum and EDTA plasma.

**Precautions and Warnings**

- All wastes should be properly decontaminated prior to disposal.
- Do not mix components from kits with different serial numbers.
- Do not use a SNAP device that has been activated prior to the addition of sample.
- WARNING:** Conjugate – H316/P332+P313/EUH208. Causes mild skin irritation. If skin irritation occurs: Get medical advice/attention. Contains Kathon. May produce an allergic reaction.

**Storage**

- Store at 2–8°C until expiration date.

**Kit Components**

Item	Reagents	Quantity
1	1 bottle of anti-chicken:HRPO/anti-NTproBNP: HRPO conjugate (Green)	3.0 mL
2	SNAP Device	5 or 10

Each SNAP device contains 0.4 mL of wash solution and 0.6 mL of substrate solution.

**Other Components:** transfer pipettes, sample tubes and reagent rack

**Sample Information**

- Samples must be at room temperature (18–25°C) before beginning the test procedure.
- Serum or EDTA plasma, either fresh or stored at 2–8°C for up to one week, can be used.
- For longer storage, serum and EDTA plasma can be frozen (-20°C or colder) and then reconstituted before use.
- Hemolyzed, icteric or lipemic samples will not affect test results.

**Test Procedure**

- Allow all components to equilibrate at room temperature (18–25°C) for at least 30 minutes. **Do not heat.**
- Hold the required transfer pipette vertically and dispense **3 drops of sample** into a new sample tube.
- Holding the bottle vertically, add **5 drops of green conjugate** to the sample tube.
- Cap the sample tube and mix thoroughly by inverting it 3–5 times.
- Place the device on a horizontal surface. Add the entire contents of the sample tube to the sample well, being careful not to splash the contents outside of the sample well.
- The sample will flow across the result window, reaching the activation circle in 30–60 seconds. Some sample may remain in the sample well.
- When color FIRST appears in the activation circle, push the activator firmly until it is flush with the device body.
- Note:** Some samples may not flow to the activation circle within 60 seconds. In this case, press the activator after the sample has flowed across the result window.
- Read test result **10 minutes** from the time of activation.

**Interpreting the Test Results**

**Abnormal Level**

If the color intensity of the sample spot is equal to or darker than the color intensity of the reference spot, NTproBNP levels are abnormal.

**Normal Level**

If the color intensity of the sample spot is lighter than the color intensity of the reference spot, NTproBNP levels are normal.

**Note:** It is possible for no color to develop on the sample spot.

**Invalid Results**

- Background**—If the sample is allowed to flow past the activation circle, background color may result. Some background color is normal. However, if colored background obscures test result, repeat the test.
- No Color Development**—If reference spot does not develop color, repeat the test.

**Percent agreement to Cardiopet**

402 samples were tested on both SNAP Feline proBNP and Cardiopet proBNP tests. Visual results from SNAP Feline proBNP demonstrated 99.5% (95% CI: 97.1 – 99.9) agreement with Cardiopet proBNP if the sample had a concentration of NT-proBNP < 100 pmol/L (n=187) and 95.0% (95% CI: 90.0 – 97.5) agreement with Cardiopet proBNP if the sample concentration of NT-proBNP was ≥ 270 pmol/L (n=140).

**IDEXX SNAPshot Dx\* Analyzer**

Test results can also be read using the SNAPshot Dx\* Analyzer. A complete description of how to enter patient data and read test results using the SNAPshot Dx Analyzer can be found in the SNAPshot Dx Analyzer user guide.

**Symbol Descriptions**

	Use by date		Authorized Representative in the European Community
	Batch Code (Lot)		Consult instructions for use
	Serial number		In vitro diagnostic
	Temperature limitation		Date of manufacture
	Catalog Number		

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\*SNAP es una marca de comercio o desposé de IDEXX Laboratories, Inc. o sus filiales en los Estados Unidos y/o en otros países. Patent information: idexx.com/patents. © 2015 IDEXX Laboratories, Inc. Todos los derechos reservados.
© 2015 IDEXX Laboratories, Inc. Todos os direitos reservados.